

Using Brava® Supporting Products with Pediatrics

Based on updated documentation guidelines, real-life use, and existing biological evaluation reports, Coloplast recently reassessed the risk to pediatrics with the use of supporting products. Updated guidelines are provided below.

Product	Caution
Brava® Protective Seal	No caution
Brava® Protective Seal Convex	
Brava® Elastic Barrier Strips	
Brava® Protective Sheet	
Brava® Lubricating Deodorant	
Brava® Skin Cleanser Wipes	
Brava® Moldable Ring	<i>This product is non-sterile and should be used with caution on premature babies or infants receiving incubator care as it may lead to increased risks of infection.</i>
Brava® Strip Paste	
Brava® Paste	
Brava® Powder	
Brava® Adhesive Remover Wipes	<i>Not recommended to be used on premature infants, neonates or infants below 4 kg (8.8 lbs) because of their immature skin.</i>
Brava® Skin Barrier Wipes	
Brava® Adhesive Remover Spray	<i>Not recommended to be used on premature infants, neonates or infants below 4 kg (8.8 lbs) because of their immature skin. Avoid direct inhalation as it may give rise to temporary irritation of the airways.</i>
Brava® Skin Barrier Spray	

Any use outside of our indication is considered off-label use and is up to the clinician’s own discretion. Clinicians must weigh the risks and benefits of available products to make an informed and appropriate decision for each patient.

