

A *unique* approach
to difficult-to-dress wounds

Triad is a sterile* coating that can be applied anywhere, adheres to wet skin, and keeps the wound covered.



Wounds on
irregular surfaces



Broken skin in the
presence of incontinence



Maceration of
periwound skin




Facilitating autolytic
debridement

TRIAD APPLICATION:



1. Always cleanse the wound before applying Triad.

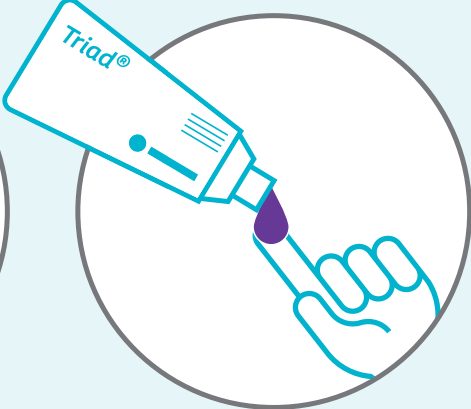

2. Triad can be applied directly from the tube or by using a gloved finger.



3. Gently spread Triad evenly over the area of application to the thickness of a dime.

Because Triad is designed for use on wet and irregular surfaces, it is **typically used on its own without a secondary dressing.**

Triad may not always dry completely and some light staining on bed linens or garments might occur.

Triad can stay in place for up to 7 days, with higher exudate levels requiring more frequent re-applications.

In the perineal area, reapply Triad after each episode of incontinence. Triad does not need to be removed completely before reapplying.


Triad instructions
for use video
Scan to view the video.




TRIAD REMOVAL:



1. Use pH-balanced wound cleanser to soften Triad.

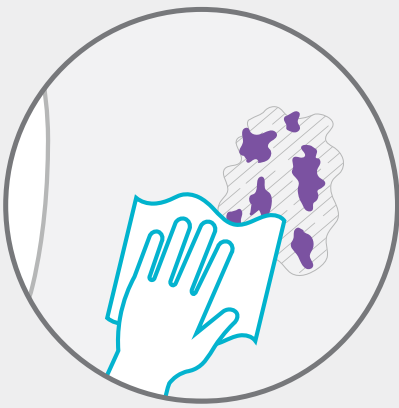

2. Gently wipe to remove without scrubbing.

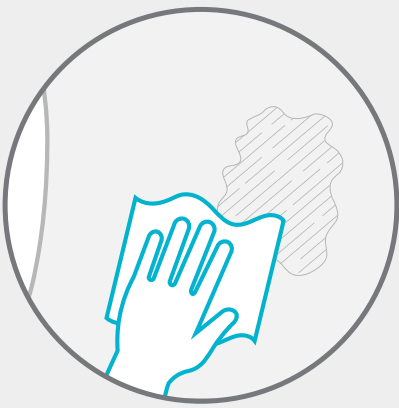

3. For complete removal repeat as needed.

Ordering Information

Triad® Hydrophilic Wound Dressing

Code	Size	Units	HCPS Code
1964	2.5 oz. (71 g)	12	A6240
1967	6 oz. (170 g)	12	A6240



*Triad is only sterile on first application.

Refer to product labeling for complete instructions including products uses, ingredients, purpose, warnings, precautions, and other information.

Coloplast United States
Minneapolis, MN 55411 / 1-800-533-0464 / www.coloplast.us

Coloplast Canada
Oakville, ON L6H 0H2 / 1-877-880-8605 / www.coloplast.ca

Coloplast and the Coloplast logo are trademarks of Coloplast A/S.
© 2025-02 Coloplast A/S. All rights reserved. PM-19840