

InterDry® improves patient satisfaction and is patient preferred over traditional therapies¹

InterDry®
Moisture-wicking fabric
with antimicrobial silver

Aim

The aim of this product evaluation is to evaluate InterDry® for skin fold management.

Methods

A total of 54 evaluations of subjects with affected skin areas on different body locations were included in this product evaluation. The product evaluation was performed in Canada between 2014-2016. InterDry was placed on the affected skin fold areas. The use of InterDry was evaluated by a health care professional on a 5-point scale with respect to ease-of-use, skin improvement after 5 days, the ability to wick away moisture, the ability to reduce odor, the ability to reduce pain, the ability to reduce discomfort and overall patient satisfaction. According to the US/Canadian instructions for use, each piece of InterDry may be used up to 5 days, depending on fabric soiling, odor, amount of moisture and general skin condition. Hence, it is assumed that this information was taken into consideration during the product evaluation.

Respondents

A total of 54 respondents have evaluated InterDry, see Table 1 for characteristics of the subjects involved. The distribution of the affected body

Product Evaluation: InterDry®
The first effective solution for skin fold management

Please evaluate your experience with InterDry

To what extent was InterDry easy to use? very easy easy average difficult very difficult

Rate improvement of skin condition after 5 days? much better better same worse much worse

How would you rate InterDry's ability to wick away moisture? very good good average poor very poor

How would you rate InterDry's ability to reduce odor? very good good average poor very poor

How would you rate InterDry's ability to reduce pain? very good good average poor very poor

Overall, how do you rate InterDry compared to your current product? much better better same worse much worse

Based on your experience, will you use InterDry in the future? Yes No

Name of evaluator: _____ Trial start date: (MM/DD/YYYY) _____ Trial end date: (MM/DD/YYYY) _____


Product Currently Used: _____ Patient Gender: male female Age: _____

Facility: Acute LTC Community Other _____ Territory Manager: _____

Comments: _____

The Coloplast logo is a registered trademark of Coloplast A/S, © 2014-2016. All rights reserved. Coloplast Canada Inc., Mississauga, ON, Canada.

www.coloplast.com Coloplast

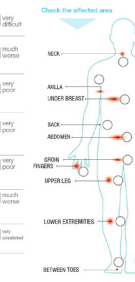

Product Evaluation: InterDry®
For skin fold management

Please evaluate your experience with InterDry

To what extent was InterDry easy to use? very easy easy average difficult very difficult

Rate improvement of skin condition after 5 days? much better better same worse much worse

How would you rate InterDry's ability to wick away moisture? very good good average poor very poor

How would you rate InterDry's ability to reduce odor? very good good average poor very poor

How would you rate InterDry's ability to reduce pain? very good good average poor very poor

Overall, how do you rate InterDry compared to your current product? much better better same worse much worse

Based on your experience, will you use InterDry in the future? Yes No

Evaluator Name: _____ Evaluation start date: (MM/DD/YYYY) _____ Evaluation end date: (MM/DD/YYYY) _____

Product Currently Used: _____ Patient Gender: male female Age: _____


Facility: Acute LTC Community Other _____ Territory Manager: _____

Comments: _____

The result of this product evaluation may be used by Coloplast for clinical R&D and/or general marketing purposes.

InterDry is a registered trademark of Coloplast A/S, © 2014-2016. All rights reserved. Coloplast Canada Inc., Mississauga, ON, Canada.

www.coloplast.com Coloplast



areas as depicted in (Figure 1). In the product evaluation, the respondents were asked about product currently used. The products currently used varied from nothing, powder and gauze to different kinds of topical product with or without antifungals.

1. Product Evaluation InterDry Canada 2w016. Doc. No. VV-020577, Version 1.0.

Patient evaluation using InterDry²

	Total	Version 1	Version 2
Number of respondents	54	36 (67%)	18 (33%)
Gender			
Male	18 (33%)	15 (42%)	3 (17%)
Female	33 (61%)	21 (58%)	12 (67%)
N/A	3 (6%)	0 (0%)	3 (17%)
Age			
21-30 years	1 (2%)	1 (3%)	0 (0%)
31-40 years	6 (11%)	5 (14%)	1 (6%)
41-50 years	0 (0%)	0 (0%)	0 (0%)
>50 years	35 (65%)	28 (78%)	7 (39%)
N/A	12 (22%)	2 (6%)	10 (56%)
Facility type			
Community care	6 (11%)	4 (11%)	2 (11%)
Acute care	24 (44%)	20 (56%)	4 (22%)
Long term care	22 (41%)	10 (30%)	12 (67%)
Other (complex care)	1 (2%)	1 (3%)	0 (0%)
N/A	1 (2%)	1 (3%)	0 (0%)
Duration of evaluation			
Range (in days)	3-31	3-31	3-28
Mean (in days)	10.4 (N=42)	10.2 (N=27)	10.7 (N=15)
Median (in days)	9.5 (N=42)	9.0 (N=27)	10.0 (N=15)


Table 1: Baseline characteristics of the included subjects

N/A: data missing/blanks/not filled out correctly or not evaluated


Affected body area	Number of respondents
Neck	2
Axilla	2
Under breast	15
Back	0
Abdomen	14
Groin	14
Fingers	0
Upper leg	0
Lower extremities	0
Between toes	1
More than 1	12

Figure 1: Distribution of affected body areas


Results

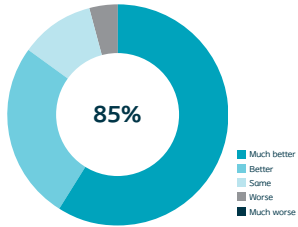


Skin condition rated as much better or better

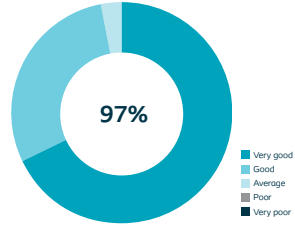


Ability to reduce discomfort rated as very good or good

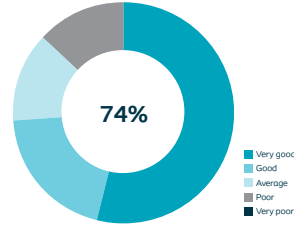


Ability to reduce pain rated as very good or good

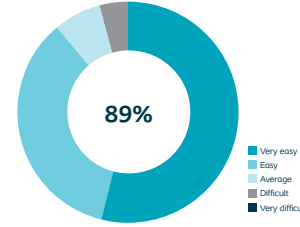


Ease-of-use as very easy or easy to use

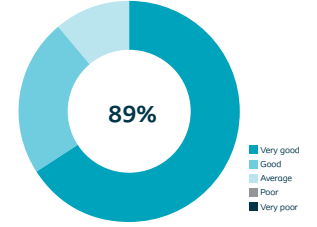


Ability to wick away moisture rated as very good or good

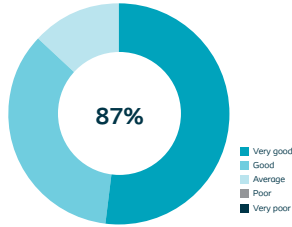


Ability to reduce odor rated as very good or good

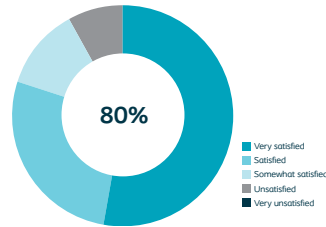

Overall patient satisfaction was rated very satisfied or satisfied

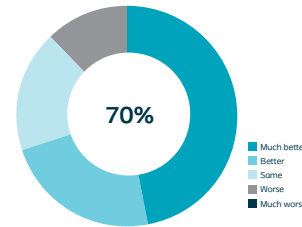

Compared as much better or better to current product

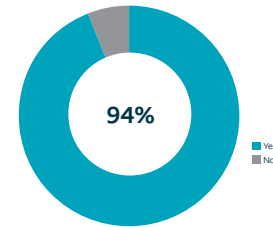

Would recommend InterDry® for future use

Refer to product labeling for complete instructions including product uses, ingredients, purpose, warnings, precautions and other information.

Coloplast United States

Minneapolis, MN 55411 / 1-800-533-0464 / www.coloplast.us

Coloplast Canada

Oakville, ON L6H 0H2 / 1-877-880-8605 / www.coloplast.ca

Coloplast and the Coloplast logo are trademarks of Coloplast A/S.

© 2024-09 Coloplast A/S. All rights reserved. PM-09280

