How to use

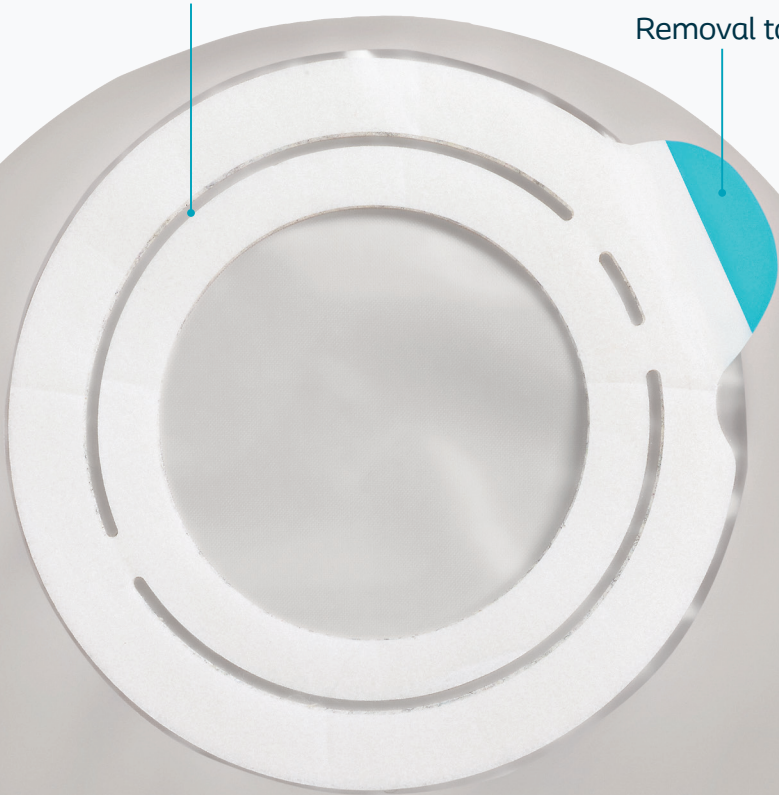
# SenSura<sup>®</sup> Mio 2-piece Flex

## Pictorial guide

Security channel

Removal tab

Prior to use, please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

SenSura<sup>®</sup> Mio

 Coloplast

# How to change your pouch

## Remove and apply a new barrier

1

### Remove your pouching system

Carefully lift the removal tab at the top of the barrier and peel downward, gently pushing the skin away from the skin barrier. Work your fingers downward as the barrier releases from the skin to help reduce risk of trauma to the skin.

**OPTIONAL:** you may choose to use an ostomy-appropriate adhesive remover like Brava® Adhesive Remover to aid in removal; do not use an oil-based product as this can interfere with the adhesion of your next barrier.

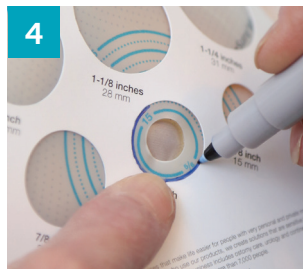
2

**Clean stoma and surrounding skin** with water and a clean cloth, removing any residue. Skin should be clean and dry before applying the barrier.

3

**Measure the stoma,** ensuring no gaps of exposed skin.


4

**Trace measurement** onto back of the barrier.


5

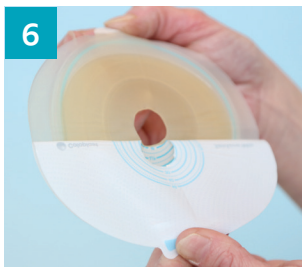**Cut opening** in the barrier.

6

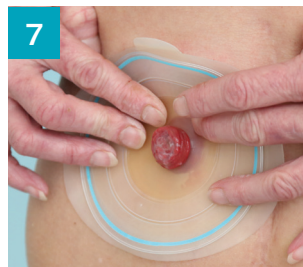**Remove protective backing** by pulling the turquoise release tab away from the barrier.

7

**Secure the barrier** by applying gentle warming hand pressure to ensure that the adhesive has made full contact with the skin. Run a finger all the way around the stoma then move outwards to the edges.

**TIP:** To minimize leakage it is important to ensure a snug fit around the stoma - focus on pressing the area around the stoma, then across the barrier surface to maximize adhesion to the skin.

For product support, call Coloplast Care at 1-877-858-2656.

## Apply the pouch

**Remove the protective backing** from the adhesive ring on the pouch.

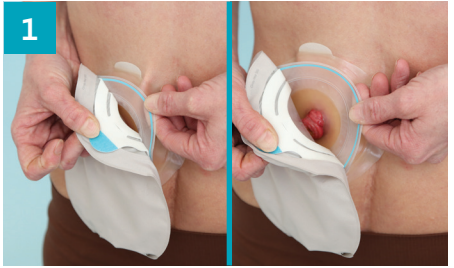
**Attach the pouch** by lining up the edge of the adhesive ring on the pouch with the turquoise line on the barrier's floating flange.

Securely connect by applying gentle pressure around the barrier coupling (you can place your fingers underneath the floating flange to pinch / seal the adhesive securely in place).

If your barrier is intact (not leaking) and you want to expel excess gas quickly or change just your pouch, here's how to **disconnect the pouch from the barrier**.


**TIP:** Make sure the adhesive ring and floating flange are both clean and dry before you reapply.

**Remove pouch** by using the turquoise tab located on the adhesive ring. Pull down and away while holding the turquoise line on the floating flange.

## Features

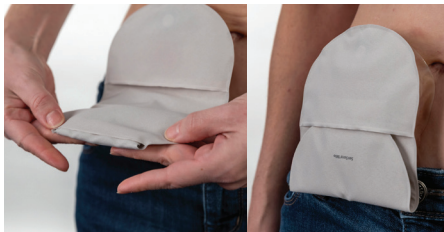**Hide-away** (drainable or high output pouches) **Filter cover**

**Hide-away:** Tuck sealed outlet up and under the protective pocket.

**For extra discretion:** Adhere the closed outlet on a gray or black drainable pouch to the VELCRO® Brand fastener dot near inspection window.

Apply one of the teal stickers in your box of pouches to the exposed part of the filter before showering, swimming or submerging in water. Remove sticker when done.

## For **drainable** pouches only

### Empty

1

**Estimating ostomy pouch contents**  
Empty when 1/3 full

**Empty** the pouch when it's 1/3 to 1/2 full\*.

\*Per the Wound, Ostomy & Continence Nurse Society guideline: Wound, Ostomy and Continence Nurses Society. (2014). *Urostomy products and tips: Best practice for clinicians*. Mt. Laurel, NJ. Author.

2

**Open the outlet** by lifting both tabs off the VELCRO® Brand fastener strip. Unfold outlet.

3

**Avoid soiling:** Fold the very end of the outlet back and connect the VELCRO® Brand dots to support easy and hygienic emptying.

4

**Empty pouch** by pinching outer edges of the outlet open. After emptying, clean the outlet.

5

**Disconnect** the VELCRO® Brand fastener dots and fold the end of the outlet up to cover the outlet opening.

6

**Seal the outlet** by continuing to roll upward, three times in total, and folding the turquoise VELCRO® Brand fastener tabs in to secure.

## For **high output** pouches only

### Empty

1

Lift up the outlet and pinch the outlet just above the plug to restrict the flow of output. Remove the plug.

Tipping the outlet up and pinching it closed ensures output stays in the pouch until you're ready to empty.

2

Continue to pinch the outlet closed as you direct it toward the toilet.

Unpinch to drain. When the pouch is empty, wipe with toilet paper and replace the plug.

VELCRO® is a registered trademark of Velcro BVBA. Used with permission.

The Coloplast logo is a registered trademark of Coloplast A/S. © 2026-03.

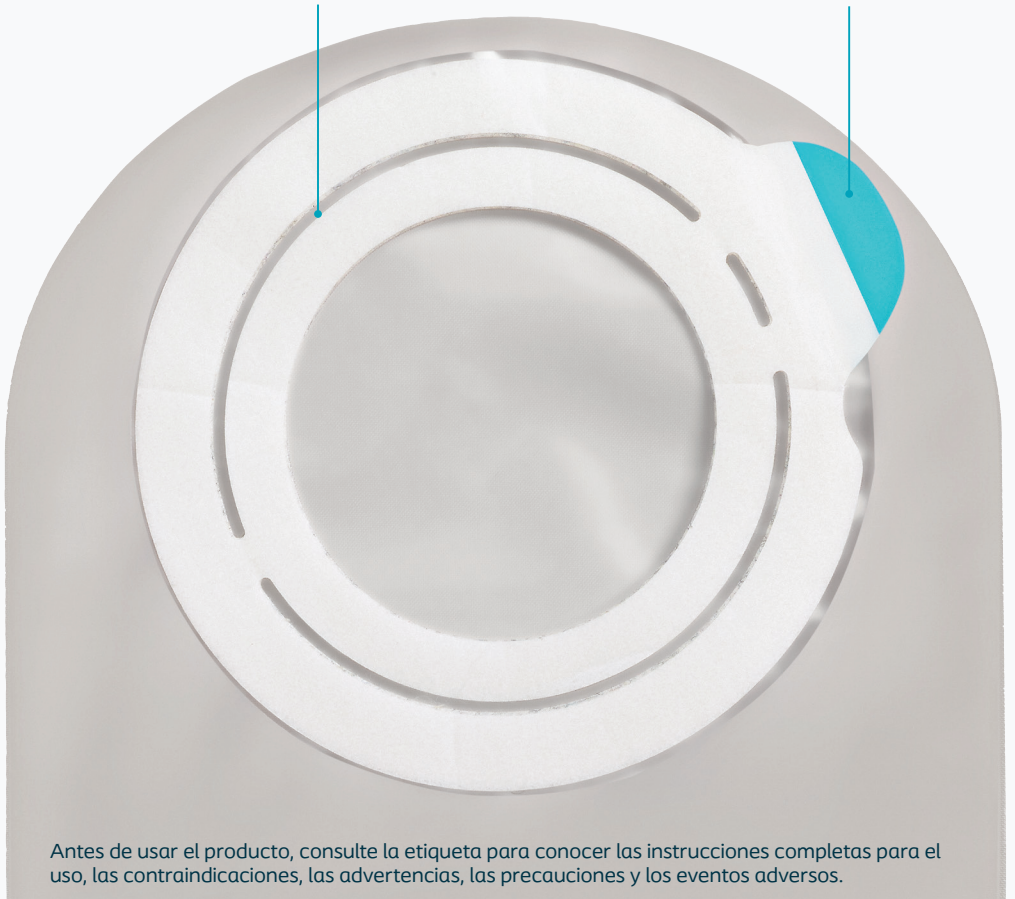
All rights reserved Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464. PM-06445 M6119N

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S. © 2026-03.
Todos los derechos reservados. Coloplast Corp. Minneapolis, MN 55411/1-800-533-0464. PM-06445 M6119N