

Basic pediatric ostomy care guidelines: kids

These guidelines provide basic information on pediatric ostomy care for nurses and other healthcare professionals working with children.

The topics covered include peristomal skin care, product selection, pouch application, and information on discharge of the pediatric patient. As a rule, adult ostomy care principles also apply to children.

However, when it comes to skin characteristics and body profiles, this particular patient group is unique and calls for special care.

We hope you will find these basic guidelines helpful. If you need more in-depth information, please consider visiting our Coloplast Professional website at www.coloplast.us/ostomy/professional/

Basic ostomy care

Removing the ostomy product

Gently release the adhesive, using a gauze and lukewarm water, and gently peel it off

Be careful when choosing adhesive removers, as they may contain noxious substances. As a rule, the use of adhesive removers should be limited and only used if:

- The epidermis would otherwise be damaged; and/or
- The removal of the product cannot be postponed

Document the reason(s) for changing the pouching system

Assessing the peristomal skin and the stoma

The skin around the child's stoma is fragile. Thus, it's important to conduct a thorough assessment of the peristomal skin as well as the stoma to identify what kind of intervention is necessary. Consider the following questions to direct your assessment:

Intact

Macerated

Red

Eroded

In case of leakage, document the area of leakage and the skin involved:

Assess whether there is leakage on the skin and/or on the back of the adhesive. Use a skin assessment tool to identify the care and the severity of peristomal skin complications

Use image documentation or keep track of any improvement or deterioration of the peristomal skin

Remember to document all interventions

If you see peristomal skin complications, identify what the cause may be:

Observe Leakage of stomal output on the skin and/or on the back of the adhesive

An allergic reaction

Mechanical damage (caused by stripping or pressure)

A pre-existing disease or condition

An infection

Assess and document the child's stoma in terms of:

Size

Protrusion (above, in level with, or below skin level)

Shape

Color

Appearance

Peristomal bulge/hernia

Assess whether the muco-cutaneous junction (stoma at the suture line) is:

Intact

Separated

Identify the characteristics of the child's effluents:

For stool, assess consistency, quantity, and color

For urine, assess clarity, presence of mucous, and odor

Cleansing the peristomal skin

Cleanse thoroughly with lukewarm water and a soft cloth

Dry gently by dabbing without causing friction

Avoid commercial wipes, as they may cause skin irritation and/or allergy

Selecting the right product

When choosing a pouching system for the child, always consider:

The child's age, weight and the shape/form of the abdominal area around the stoma (body profile)

The location and type of the stoma:

- ileostomy, colostomy, urostomy
- stoma above, in level with, or below skin level

The consistency and daily volume of the effluent

The child's mobility and activity

Applying the pouching system

Warm the pouch adhesive with your hands, but do not heat the product with an external heat source (e.g. hair dryer)

When the adhesive barrier has been applied, apply gentle pressure to the adhesive to fully activate the adhesion to the skin

Ensure a tight seal around the stoma. If necessary/possible, use a stoma measuring guide to get the right size hole in the adhesive

If it's difficult to obtain a tight seal, consider using supporting products/accessories such as a moldable ring, strip paste, or stoma paste.

Wear-time of ostomy product

A minimum of 24 hours, to prevent disrupting the integrity of the peristomal skin.

Up to four days, depending on the integrity of the peristomal skin.

Discharge of patient

Pre-discharge

Make sure to provide all necessary information on:

Training in ostomy care

Stoma and peristomal skin complications

The selected pouching system/accessories

Prescription for the pouching system

Contact with a supply company

Financial resources

How and when to reach the ostomy nurse or surgeon (provide relevant phone number(s) and email address(es).
• The date/time for the follow-up appointment

Inform parents specifically on:
• which clinical signs of complications to watch out for (for example, prolapse of the stoma or peristomal skin complications such as eroded skin)

Post-discharge

Recommendations for follow-up contact:

Start with weekly follow-up calls with the parents

At the follow-up appointment with the surgeon, it is advised that the ostomy nurse is present

Use online applications if the family lives far away from your healthcare center (remember to check that local regulations allow for this)

The materials and resources presented are intended to be an educational resource and presented for general information purposes only. They are not intended to constitute medical or business advice or in any way replace the independent medical judgment of a trained and licensed physician with respect to any patient needs or circumstances. Please refer to product 'Instructions for Use' for intended use and relevant safety information. Considerations, techniques, practice, use, and outcomes may vary based on clinical practice and judgment.