

Introducing the new

SpeediCath[®] *Soft*

for men

NEW
soft
catheter



The truth is:

The whole process
should be quick


SpeediCath® Soft is designed to make catheterization simple and easy. It is pre-lubricated so it is ready to go when you are.

This means minimal steps for preparation, as it does not require extra handling steps like having to lubricate the catheter yourself.


Instantly
ready-
to-use



Get off to a good start ...

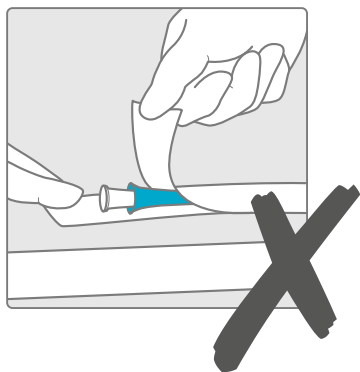

Hold the packaging upright when opening.

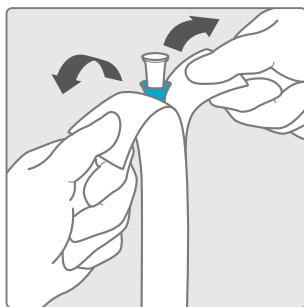

1


Only open the top part of the packaging. To ensure the quality and hydration of the coating, SpeediCath® Soft is stored in a water-based solution.



Scan this code with your smartphone or tablet to watch a short video about how to use SpeediCath Soft or visit coloplast.to/SpeediCathSoft




... with *SpeediCath® Soft*

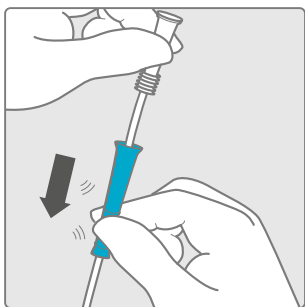


2

Release the insertion grip and pinch the wavy section.

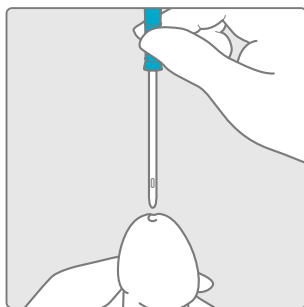


3

Use the insertion grip to insert the tip of the catheter.

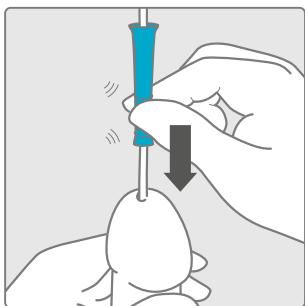

4

Once the tip is inserted in the urethra, use a pinch-release action to insert.

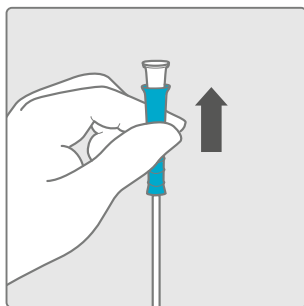

5

Push the insertion grip back onto the connector and pull catheter out.



Before use, always consult the 'instructions for use' document delivered with the products.


The truth is:

A good grip matters

SpeediCath® Soft has an insertion grip made from a soft, flexible material. It is designed to ensure a confident grip of the catheter, so you can easily insert it without touching the tube or tip.

Practice the pinch

- Grasp the insertion grip on the wavy section of the lower half.
- Release the pressure on the insertion grip to move it upwards on the catheter.
- Repeat the pinch-release action to insert the catheter.

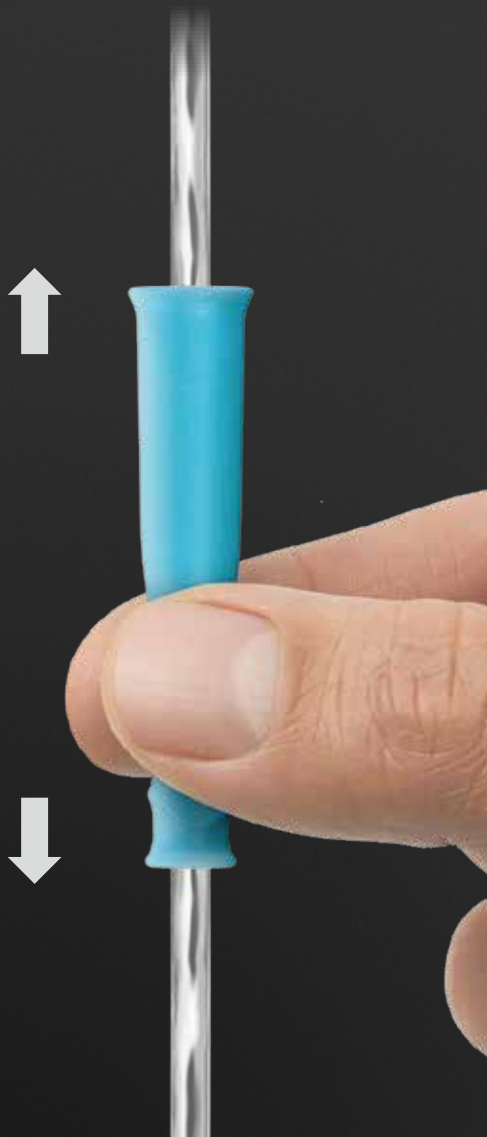

Did you like your *samples*?

Getting SpeediCath® Soft is simple

Contact your catheter supplier using the information below.

SpeediCath Soft - a hydrophilic straight tip catheter for men

Product code	Size	Catheters per box	Catheter length
27010	● 10 FR	30	13 in
27012	○ 12 FR	30	13 in
27014	● 14 FR	30	13 in
27016	● 16 FR	30	13 in

If you need help contacting your supplier, or have questions related to using a catheter, contact Coloplast® Care at:



1-866-226-6362



bladder.coloplastcare.us



care-us@coloplast.com

SpeediCath® Soft is indicated for use by patients with urine retention and patients with post void residual volume (PVR) due to neurogenic and non-neurogenic voiding dysfunction. The catheter is inserted into the urethra to reach the bladder allowing urine to drain. The product is for adult male patients only. SpeediCath catheters are available by prescription only. Patients performing self-catheterization should follow the advice of, and direct questions about use of the product to, their medical professional. Before using the device, carefully read the product labels and information accompanying the device including the instructions for use which contain additional safety information. The SpeediCath product is for single-use only; discard it after use. If you experience symptoms of a urinary tract infection, or are unable to pass the catheter into the bladder, contact your healthcare professional. The risk information provided here is not comprehensive. To learn more, talk to your healthcare provider. For further information, call Coloplast Corp. at 1-866-226-6362 and/or consult the company website at <http://www.coloplast.us>

[Ostomy Care](#) / [Continence Care](#) / [Wound & Skin Care](#) / [Interventional Urology](#)



Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464

www.coloplast.us The Coloplast logo is a registered trademark of

Coloplast A/S. © 2020 Coloplast Corp. All rights reserved.

PM-12665 09.20

Coloplast® Care

Connect to confidence

Coloplast Care is a patient support program that guides you to a better life through simple, straightforward advice about bladder management. We offer a helping hand whenever you need support along with tips and tools you need to take control of your bladder issues.

Sign up for Coloplast Care online at:

 bladder.coloplastcare.us



Want to learn more about *SpeediCath® Soft?*



Visit our website to learn more about SpeediCath Soft at coloplast.us



Watch videos about the benefits of SpeediCath Soft at coloplast.to/SpeediCathSoft

Questions?



Contact your supplier today to try a sample of the new SpeediCath Soft catheter for men.

