

SenSura[®] Mio 1-piece urostomy


Pictorial guide

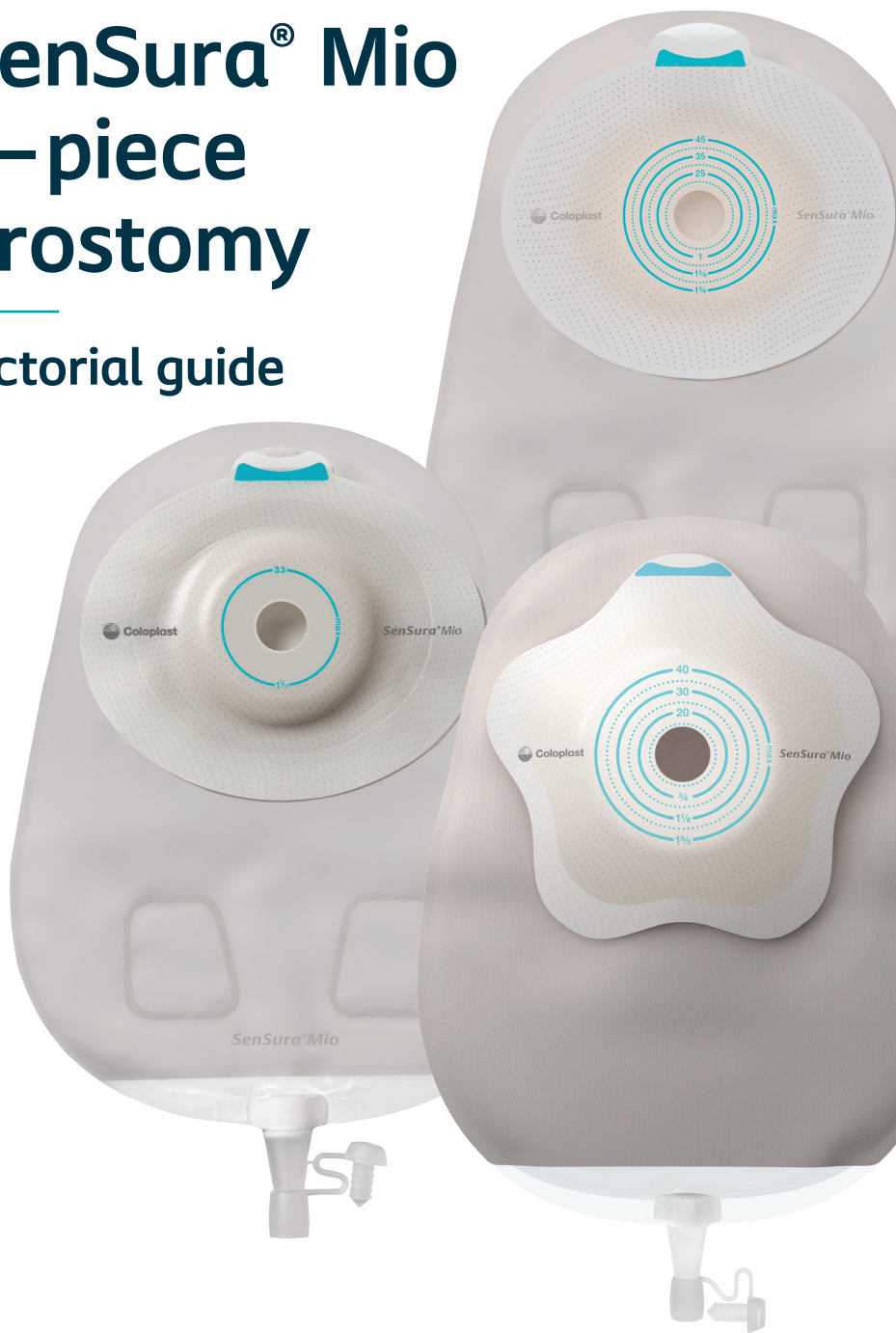
Prepare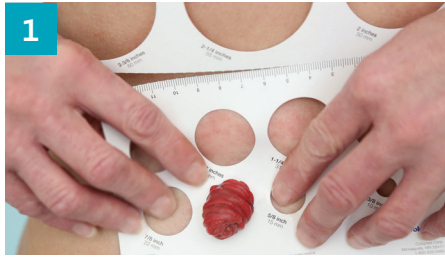

Measure the stoma.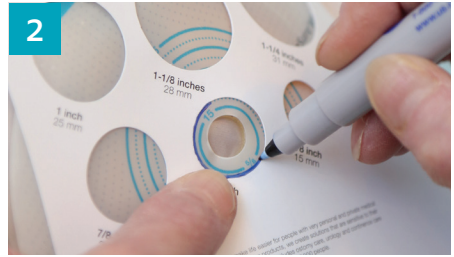

Trace measurement onto back of the barrier.

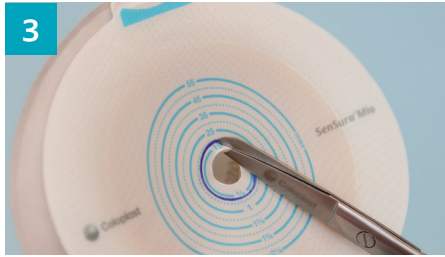Cut opening in the barrier.

Remove protective backing by pulling the turquoise release tab away from the barrier.

Apply

Clean stoma and surrounding skin with water and a clean cloth, removing any residue. Skin should be clean and dry before applying the barrier.

Before applying the pouch, check to ensure the outlet has been securely fastened (plug to close). Apply pouch around the stoma.

Secure the barrier by applying gentle warming hand pressure to ensure that the adhesive has made full contact with the skin. Run a finger all the way around the stoma then move outwards to the edges.

Empty

1

Estimating urostomy pouch contents

Empty when 1/3 full

Empty or remove the pouch when it's 1/3 to 1/2 full*.

2

Empty the urostomy pouch by holding the end of the outlet upwards. Pinch spout to control drainage and remove the plug. Direct the spout and release to drain.

3

The Coloplast urostomy night bag can be attached for bedside drainage. If using a different drain bag, you may need an adaptor.

Remove

1

Gently remove the barrier by pulling the white removal tab away from the abdomen while gently pushing on the skin to help with removal.

Scan the QR code to view the how to use video

For product support, call the Coloplast Consumer Care Team at 1-877-858-2656.

Prior to use, please refer to product labeling for complete product instructions for use, contraindications, warnings, precautions and adverse events.

*Per the Wound, Ostomy & Continence Nurse Society guideline: Wound, Ostomy and Continence Nurses Society. (2014). *Urostomy products and tips: Best practice for clinicians*. Mt. Laurel, NJ. Author.

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El logotipo de Coloplast es una marca comercial registrada de Coloplast A/S. © 2023-10. Todos los derechos reservados Coloplast Corp. Minneapolis, MN 55411 / 1-800-533-0464. PM-03213 M6121N