

SenSura® Mio Kids

1-piece Urostomy pouch



Prepare

1



Measure the stoma using the stoma measuring guide



Using the measuring guide, trace and draw the shape of the stoma on the barrier



Use a pair of small curved scissors when cutting the hole in the barrier



Remember to close the outlet before applying the pouch



The outlet can be hidden in the pocket on the front of the pouch



Thoroughly clean the stoma and the surrounding skin. The skin must be completely clean, dry, and free of oily substances

Apply

2



Peel off the release liner and discard it safely out of reach of children



Align the barrier around the stoma. To prevent any harm, be sure that the barrier does not touch the stoma



Begin applying pressure in the area around the stoma and lightly press the adhesive



Run a finger all the way around the stoma to ensure that the adhesive seals



Apply light pressure on the pouch

Empty

3



Hold the plug of the outlet upwards and squeeze the outer end of the outlet



Pull out the plug and relax your hold on the outlet to drain the pouch



After complete drainage, ensure that the outlet is firmly closed with the plug fully inserted



Consider using a bedside drainage bag during sleep



Push the narrow end of the connector of the bedside drainage pouch into the outlet of the urostomy bag



The other end is connected to the bedside drainage bag

Remove

4



Gently pull the removal ear away from the skin



Peel the adhesive slowly downwards



Tip: Use Brava® Adhesive Remover to gently remove the barrier

Prior to use, refer to the product 'Instructions for Use' for intended use and relevant safety information.