

# SenSura® Mio Kids

## 2-piece Flex Urostomy pouch



### Prepare

1



Measure the stoma using the stoma measuring guide



Using the measuring guide, trace and draw the shape of the stoma on the barrier



Use a pair of small curved scissors when cutting the hole in the barrier



Remember to close the outlet before applying the pouch



The outlet can be hidden in the pocket on the front



Thoroughly clean the stoma and skin. Ensure the skin is clean, dry, and free of oily substances

### Apply

2



Peel off the release liner and discard it safely out of reach of children



Align the barrier around the stoma. Ensure the barrier doesn't touch the stoma



Apply light pressure to the barrier to ensure fast adhesion



Run a finger all the way around the stoma to ensure that the adhesive seals



Ensure the barrier is dry and clean



Remove the protective film from the adhesive ring of the pouch



Align the bottom of the pouch coupling with the flange located on the barrier



Apply light pressure on the pouch and barrier




Start from the bottom and smooth the coupling upwards

## Empty


3



Hold the plug of the outlet upwards and squeeze it

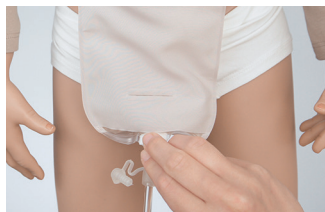

Pull out the plug and relax your hold on the outlet to drain the pouch

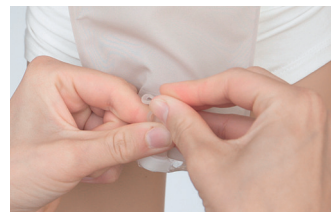

After complete drainage, ensure that the outlet is firmly closed with the plug fully inserted



Consider using a bedside drainage bag during sleep



Push the narrow end of the connector of the bedside drainage bag into the outlet



The other end is connected to the bedside drainage bag

## Remove

4



**Remove the pouch:** Apply light pressure to the flange of the barrier with one hand



Remove the pouch with the other hand by pulling the tab on the pouch slowly



**Remove the barrier:** Gently pull the removal ear away from the skin



Peel the adhesive slowly downwards



**Tip:** Use Brava® Adhesive Remover to gently remove the barrier

Prior to use, refer to the product 'Instructions for Use' for intended use and relevant safety information.

Ostomy Care / Continence Care / Wound & Skin Care / Interventional Urology

